

3M™ Bair Paws™ System  
Warming Unit Model 875



# Versatile Patient Warming



The 3M™ Bair Paws™ system is revolutionizing forced-air warming by providing clinicians options for comfort and clinical warming throughout the perioperative process in one convenient gown. When connected to a Bair Paws warming unit, the Bair Paws gown provides clinical prewarming or comfort warming as patients await surgery. In the O.R., the same gown offers clinical warming for select surgical procedures when connected to a 3M™ Bair Hugger™ warming unit. The gown continues to warm in PACU, with both comfort and clinical capabilities.



## Mounting configurations:



**Wall Mount**  
90080



**IV Pole Mount**  
90076



**Bedrail Mount**  
90079



**Wall-to-Rail Mount**  
90074




Only the 3M™ Bair Paws™ system offers patient-adjustable warming with a simple twist, as the unit's handheld controller allows patients to adjust the temperature of the air flowing through the gown to a level that's right for them.

The warming unit model 875 is lightweight, simple to use and is placed easily on the wall, bedrail, IV pole or a flat surface.

## The Bair Paws warming unit provides:

- The ability to prewarm patients to help in the prevention of unintended hypothermia
- Personalized comfort as patients control the unit's airflow and temperature (ambient to 43°C)
- Easy "snap-fit" flexibility in mounting the unit on the wall, a bedrail or an IV pole. The unit can also be placed on a flat surface.
- Convenient, built-in hose storage
- Small, lightweight design

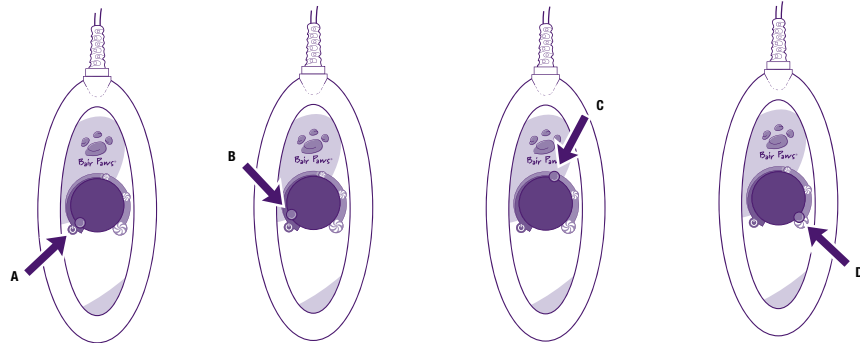

When the knob is in position A, the warming unit is OFF.

As the knob is rotated from position A to B, the warming unit turns on LOW AIRFLOW and LOW HEAT.

By turning the knob from position B to C, the air temperature increases from slightly above room temperature to approximately 41°C (106°F), while the blower remains on LOW AIRFLOW.

As the knob rotates clockwise past position C, the air temperature and blower's airflow both increase.

At position D, the warming unit is on HIGH HEAT and HIGH AIRFLOW. The air temperature will stabilize at 43° ± 3°C (109° ± 5.4°F).

## Specifications

**Dimensions**  
13" high x 7.7" wide x 4" deep  
(33 cm high x 19.6 cm wide x 10.2 cm deep)

**Weight**  
7.0 lb (3.2 kg)

**Operating Temperatures**  
User adjustable from ambient to 43° ± 3°C

**Filter**  
Dust filter

**Device Ratings**  
110-120 VAC, 50/60 Hz, 4.6A  
220-240 VAC, 50/60 Hz, 2.8A

## Gown Sizes and Configurations

**Pediatric**  
33" long; 40" sweep

**Small**  
44" long; 55" sweep

**Standard**  
51" long; 64" sweep

**X-Large**  
51" long; 122" sweep

Gown kits include patient warming gown, bonnet, booties, personal belongings bag and shoe bag.



## Ordering Information

**Patient Warming Gown**

81501	Pediatric	20/case
81001	Standard	30/case
81101	Small	20/case
81201	X-Large	20/case

**Patient Warming Gown with Booties**

83501	Pediatric	20/case
83001	Standard	30/case
83101	Small	20/case
83201	X-Large	20/case

**Patient Warming Gown Kit**

84501	Pediatric	20/case
84001	Standard	30/case
84101	Small	20/case
84201	X-Large	20/case

**Patient Booties**

90065	Std. Booties	30 pairs/case
90091	XL Booties	30 pairs/case



**Arizant Healthcare Inc.,**  
a 3M company  
10393 West 70th St.  
Eden Prairie, MN 55344 USA  
Phone 800-733-7775  
Fax 800-775-0002  
www.bairpaws.com

**For more information please contact your 3M Patient Warming representative at 1-800-733-7775 or visit [www.bairpaws.com](http://www.bairpaws.com).**

3M is a trademark of 3M Company, used under license in Canada. BAIR PAWS, BAIR HUGGER and the BAIR PAWS logo are trademarks of Arizant Healthcare Inc., used under license in Canada.  
©2012 Arizant Healthcare Inc. All rights reserved. 602419C 10/12